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**Put the mask on your face,** with the nose clip on your nose.  
**Position the elastic bands around the back of your head,** on both sides of the ears without crossing them.  
**Check** that the mask covers your face properly (from your nose to your chin).

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**Pinch** the nose clip so that it hugs the shape of your nose.  
**Test the seal.** Refer to the section: *Performance test: check the seal of the mask.*  
**Remove** the mask without touching the front of it, moving it away from your face and clothes.

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**The respiratory protection mask FFP2 NR is category III personal protective equipment in accordance with the requirements of regulation 2016/425. It is designed to protect the respiratory health of the user. This product has a filtration efficiency against particles of at least 94%.**

- Test de performance: vérification de l'étanchéité du masque**
- Couvrir la surface filtrante du masque en utilisant une feuille plastique maintenue en place avec les deux mains.
  - Inhale: the mask must fall slightly against the face.
  - Then exhale: the mask must bulge outwards.
  - If the mask is not flat against the face, this is an indication that it is not impermeable and must be readjusted. Tighten the nose clip and reposition the elastic bands for a better adjustment.
  - Then follow steps 1 to 3 again to ensure that the mask is impermeable.
  - After several unsuccessful attempts, change the model as it is inadequate.

- Désignation**
- FFP2 NR\* respiratory protection mask. CE 0082\*\* certified. Meets the requirements of European Standard EN 149: 2001+A1: 2009 drafted in relation to Regulation (EU) 2016/425 on personal protective equipment.. CE marking in accordance with the requirements of regulation 2016/425 relating to personal protective equipment.
  - The mask is designated as a "filtering half-mask against particles": PFE\*\*\*>94%, 0.6µm.
  - Made in France, at the Combaude site, Clermont-Ferrand (Puy de Dôme).
  - You will find the mask documentation (technical sheet, order form, EU declaration of conformity) on our website: protective.michelin.eu**

- Description**
- Respiratory protection mask that folds and is made up of non-woven polypropylene.
  - Does not contain any components made of latex, PVC or silicone.
  - Two elastic loops to go around the head to keep the mask perfectly in place on the user's face.
  - A metal and flexible nose clip to adjust the mask around the nose for greater comfort and sealing by ensuring a hold on the face.

- Précautions particulières**
- USAGE UNIQUE: ne pas laver le masque
  - Do not use the mask if the packaging is open or damaged.
  - Contaminated masks must be properly disposed of according to the regulations in your country.
  - Discard and replace the mask if it is damaged and beyond the limits set by the working conditions. The mask must not be worn for longer than 8 hours.

- Avertissement et limites d'utilisation**
- Ensure the manufacturer's instructions are carefully followed.
  - Incorrect use or incorrect fitting of the mask or failure to comply with the recommended duration of wear can reduce the effectiveness of the mask and can affect the user's health.
  - DO NOT USE the mask, nor enter or stay in a contaminated area in the following situations :
    - If the concentration of oxygen in the air is below 17% or is likely to drop to that level (work in enclosed and confined space).
    - They provide protection against aerosols, but are ineffective against vapors or gases.
    - In explosive atmospheres.
  - Do not modify the mask or misuse it.
  - Wearing the mask if you have a beard could prevent a good seal around the face, thereby creating a leak.
  - This mask has not been designed for use by children. (Medium-sized mask)
  - The protective breathing masks must be chosen in accordance with the type and concentration of the dangerous substances.
  - If you experience breathing difficulties, dizziness or other symptoms, leave the contaminated area immediately.
  - The product is manufactured in such a way that one of its constituent parts breaks or tears, thus eliminating the danger associated with the risk of being caught.

- Conditionnement**
- 1 mask per blister.
  - Box of 50 masks, 50 blisters of 1 mask.

- Conditions de conservation et de stockage**
- Before use, keep the masks in their original packaging, away from contaminated areas, dust, sunlight, extreme temperatures (storage temperature between 5°C and 35°C), excessive humidity and harmful chemical products. The mask has a 2-year shelf life in its sealed plastic pouch from the date of production.

- Instructions avant utilisation**
- Wash your hands before putting on the mask and after removing it.
  - Use this product immediately after opening the packaging.
  - Check that the mask is in perfect condition before use (check that the nose clip, elastic loops and the filtration material are in good condition and are not damaged).
  - Do not use a degraded mask and/or if the storage period has expired.

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